AXI

Wherever you want 2 go

Quality and style

Axi transport chair

Are you looking to provide your mobility-impaired visitors with a comfortable, user-friendly chair? Then Axi is the answer! A vandalism and theft-proof transport chair that is just right for your public areas thanks to a unique nesting system. With their contemporary design, these striking Axi transport chairs will boost your image as a modern, service-oriented organisation.

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Great space-saver

Unique nesting system*

Transport chairs that are used to take visitors from the car park to the building entrance, for example, can often cause inconvenience. People leave them everywhere except where they belong. To tackle this problem, Axi transport chairs can be stored in an Axi park that can be easily identified by everyone. The chairs can be nested neatly inside each other. The nesting system prevents chaos in your public spaces.

* Patented

Axi park

The Axi park is easy for everyone to identify.

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Varied colour range

What colour fits in with your organisation?

The Axi is available in all RAL colours, so you can choose a colour that matches your own house style. Boost your image with the Axi and brighten up your reception area.

The coating

The highly corrosion resistant cathodic coating makes the Axi and all its fastening materials weatherproof.

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Easy to use

Axi in 6 steps

1. If necessary, insert a coin in the coin slot and release the Axi.
2. If present, use the luggage carrier so you can keep your hands free.
3. Start the Axi by pressing the black brake lever downwards.
4. The occupant can now sit down. Before you start, the feed should be on the footrest.
5. Tackle any obstacles using the tilt pedal.
6. After use, return the Axi to the Axi park and retrieve your coin.

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Safe to use

Everything under control

The brake lever engages the Axi brake. As the accompanying person, you therefore have full control over the transport chair. But besides control, stability is also an important factor. Because the wheels are optimally balanced, inexperienced users can use the Axi both indoors and outdoors with no problems at all. The chair is also easy and safe for the occupant to get in and out of.

NB: For safety reasons, Life & Mobility recommends using a seat belt when using the chair outdoors.

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Comfort seating

Focus on the end user

The user of a transport chair wants to sit comfortably even if they are only using it for a short time. That is why the Axi is fitted with a preformed seat, backrest and armrests. This combination delivers surprisingly good sitting comfort. For even greater comfort, we also offer optional 3D-woven seat and backrest covers.

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AXI_{2GO}

Durable and vandalism-proof

These days, transport chairs not only have to be durable and low maintenance, but - sadly - they also need to be vandalism-proof. So the frame, seat and backrest of the Axi are made entirely of high-grade steel with a minimum number of moving parts. The Axi is fitted with maintenance-free, anti-leak tires, so the chair is always light to manoeuvre and easy to steer. Thanks to the protective powder coating, the Axi can be used both indoors and outdoors.

Hard to steal

The optional coin slot and/or anti-theft post significantly reduce the risk of theft. Tip: You can decorate both the Axi itself and the anti-theft post with a printed design of your own choice.

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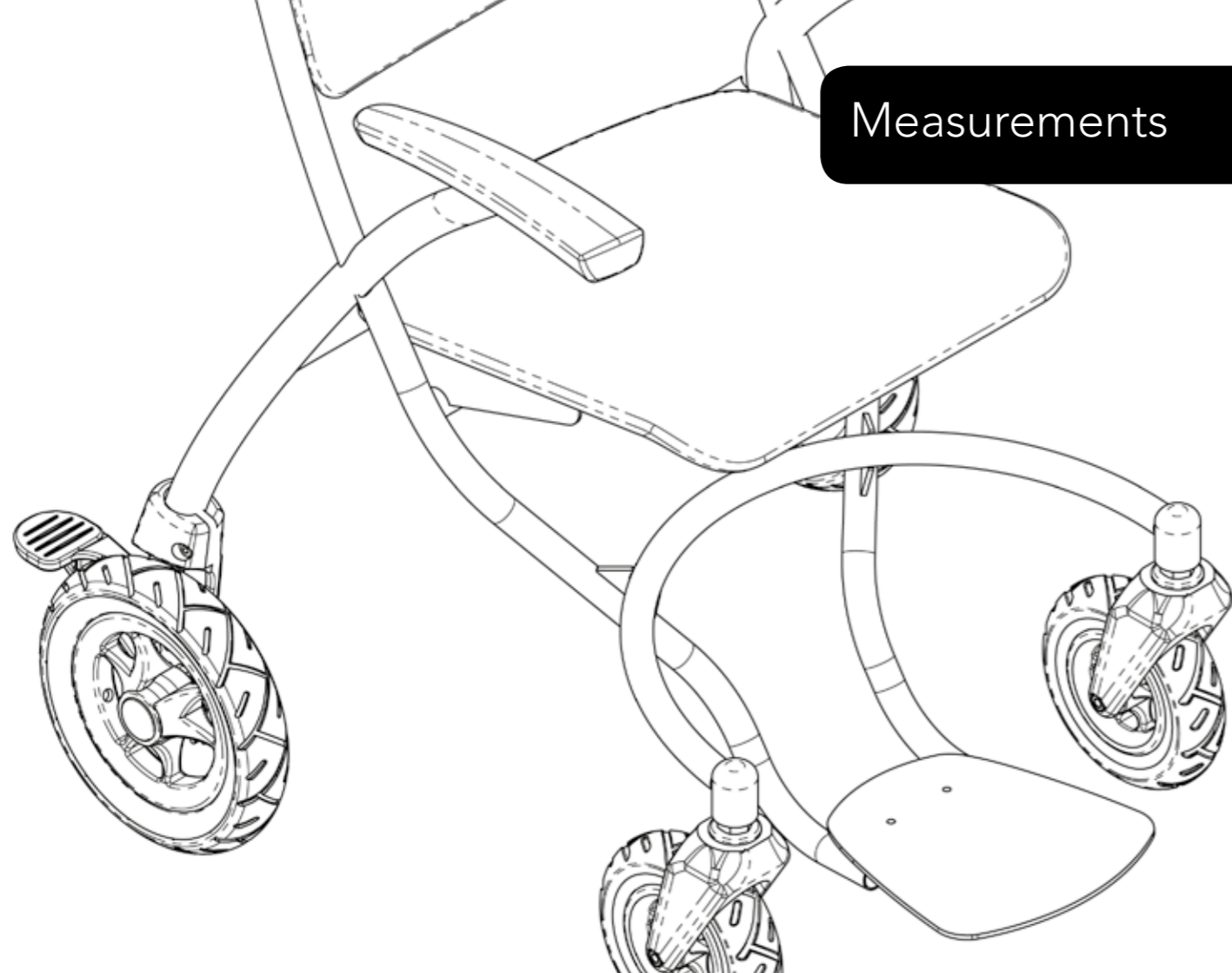
Accessories

Coin lock / master coin
Muntslot / dummy munt

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