SIMPLY[®]

SIMPLY[®]

Funktionelt understel Functional undercarriage

Passiv siddestilling
Passive seating

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The SIMPLY seat shell undercarriage convinces through its flexibility, the specific construction as well as its versatile use. In care with seat shells and modular seating systems for children, undercarriages are obligatory for indoor and/or outdoor areas. Their purpose is to ensure the mobility and the transport of the insured party. SIMPLY fulfils these aims one hundred percent.

Flexibility

In its standard version, SIMPLY is equipped with a trapezoid receptacle, which permits the accommodation of all interco seat shells (MAGICLIGHT, MAGICLINE, ORTHOLIGHT and ERGOLINE passive) quickly and without problems. Seat shells from other manufacturers can be accommodated on request, for example by means of an intermediate adapter. SIMPLY has continuous inclination adjustment (seat tilting) up to 35° via gas-pressure spring(s), which takes particular account of individual mounting requirements of patients. In this respect, the height of the seat in the knee area does not alter as a result of the tilting via a pivotal axle at the front.

HMV-nr.: 26.99.01.1006 (12"/16" hjul/wheels)
26.99.01.3005 (22"/24" hjul/wheels)

Forskellige størrelser for drivhjul (her 12")
Various sizes for the back wheels [here: 12"]

Sammenklappet mhp. transport
Folded up for transport

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SIMPLY is equally well suited for both small and large patients. The maximum accommodation of 80 kg with the standard seat width of 45 cm permits individual equipping of the seat or mounting unit. Frame stabilisers are possible in addition. The flexible, adjustable pushing grip helps ensure that no restrictions are caused for the accompanying person.

Construction

The basic construction of SIMPLY is rigid; seat widths of 35 cm, 40 cm and 45 cm are standard. Oval aluminium tubing, equipped with an environmentally friendly and permanent synthetic coating, provides the necessary stability. SIMPLY can be folded up quickly for transport with just a few turns of the hand. The SIMPLY frame construction is based on the modular principle so that, amongst other things, the necessary flexibility is also given during re-use. Seat width and wheel sizes can be converted at any time.

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The SIMPLY wheel receptacles are designed such that both small rear wheels (12" or 16") as well as large rear wheels (22" or 24") can be assembled without problems. As a fundamental rule, drum brake wheels with full floating axles are envisaged, which are operated via the brake handles on the pushing bar. In their standard version, the front wheels are designed as 200 x 50 mm pneumatic tyres. Upon request, all wheels can also be designed as accident-proof.

Areas of use

SIMPLY seat shell undercarriages are very versatile in terms of their use. SIMPLY can be used both indoors as well as outdoors. Thanks to the compact construction and manoeuvrability, SIMPLY fulfils almost all requirements both indoors and outdoors.

Active patients, cared for with an ERGOLINE seat shell, can benefit from SIMPLY thanks to the continuous adjustment of the seat shell receptacle. Passive patients, cared for with a MAGICLIGHT, MAGICLINE, ORTHOLIGHT and ERGOLINE passive, can participate in social life at all times thanks to the mobility gained. In special designs, SIMPLY enables the accommodation of the most differing mounting systems at any time, which can be prepared in accordance with individual requirements. The newly available abduction frame of SIMPLY offers almost unlimited accommodation possibilities for seating and mounting systems. Flexibility, construction and areas of use make SIMPLY seat shell undercarriages indispensable companions in the care of patients with seat shells and mounting systems.

We recommend that you select the suitable technical aids for your child together with your specialist doctor, therapist and with a specialist medical equipment dealer of your choice. We at interco shall be pleased to provide you with assistance on site. Questions? Simply give us a call – we shall be glad to help.

Medicinske krav Medical requirements

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Support of the therapeutic effect of a seat shell/of a seating system, for example in terms of the seat mounting and angle adjustment or even in terms of individual mobility.

Your personal, local specialist advisor will assist you in the planning of an individual solution, taking account of your personal requirements. We at interco guarantee the availability of replacement parts or readjustment over a period of years.

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Certified safety

The product combination undercarriage SIMPLY® with seating systems MAGICLIGHT®, MAGICLINE® or ORTHOLIGHT® with a wheelchair restraint system Protector is successfully tested according to ISO 7176-19 and ISO 10542-2. This product combination in combination with a restraint system is suitable for patient transport in a motor vehicle, when the vehicle is equipped with an appropriate device for fastening.

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Basic features

- ▶ Seat-shell receptacle trapezoid, adjustable depth
- ▶ Seat tilting via a gas-pressure spring 300 N (with two gas-pressure springs when seat width larger than 45 cm)
- ▶ Back angle adjustment/seat tilting continuously up to a maximum of 35°
- ▶ Height and angle-adjustable pushing grip
- ▶ Drum brake wheels, alternatively 12", 16", 22" or 24" with quick-release axle
- ▶ Parking brake handle with three locking positions
- ▶ Swivelling support rollers
- ▶ Integrated wheel-base extension
- ▶ Maximum load for SIMPLY: 80 kg (including patient's weight and seat unit with accessories)
- ▶ SIMPLY with individual measurement of seatdepth or seatwidth according to the needs and the measurements of the special seating system, ask for quotation

HMV-nr.: 26.99.01.1006 (12"/16" hjul/wheels)
26.99.01.3005 (22"/24" hjul/wheels)

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Status as per August 2014
Subject to technical alterations
without prior notification

Certificeret ifølge
Certified as per
DIN EN ISO 9001:2008
Reg.-nr. 73 100 357-1
DIN EN ISO 13485:2007
Reg.-nr. 70 105 357-1
TÜV Hessen

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© interco GmbH, Eitorf, 2014